
Escali

PRODUCT
MANUAL

Body Fat / Body Water Bathroom Scale

Model # BFBW200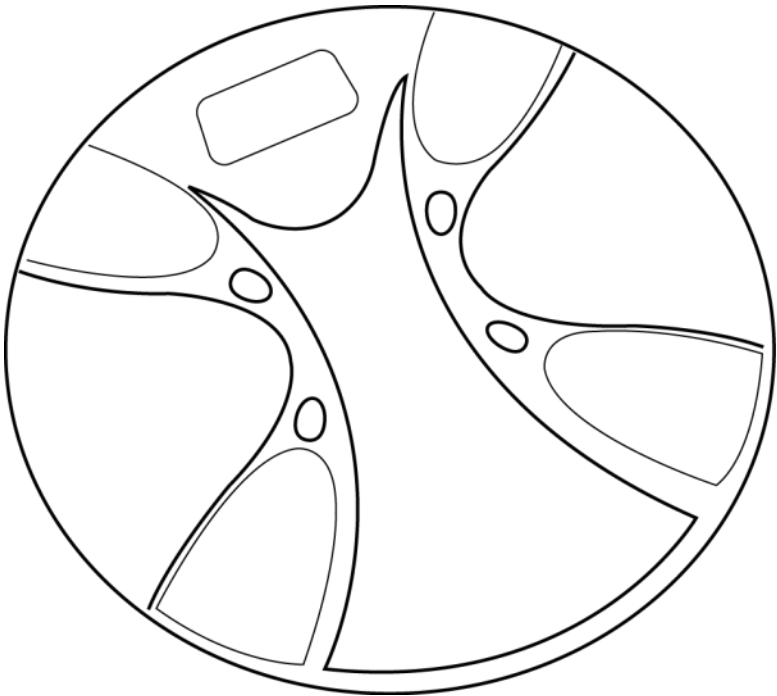

www.escali.com

Thank you for choosing the Escali BFBW200 Body Fat / Body Water Bathroom Scale. This state-of-the-art, precision measuring instrument is only accurate when used on hard, flat and level surfaces. This scale will not function correctly on carpet. The electronic sensors are very sensitive; avoid dropping or jarring the scale, and please keep it where it will be protected from impact.

Warning! Do not use this product if you have a pacemaker or other internal electronic medical devices.


INITIAL SETUP: CREATING & EDITING USER PROFILES

1. Remove the insulating plastic strip from the battery compartment. Ensure the batteries are properly seated within the +/- terminals.
2. Use the **Kg St Lb** switch on the base of the scale to selected your desired unit of measure; kilograms (kg), stones (st) or pounds (lb).
3. To set up a new user profile (or edit an existing profile) press the button and use the or arrow buttons to select a user profile number (1-10.) Press to confirm.
4. Select your gender using the or arrow buttons and press to confirm.
5. Select your height using the or arrows and press to confirm.
6. Select your age using the or arrows and press to confirm.
7. When the display shows a dashed line, step on the scale with clean and dry bare feet. The scale will measure your weight and then calculate your body composition.

Note: This body scale can save up to 10 unique user profiles. To program additional profiles, follow steps 3 through 7.

DISPLAY DIAGRAM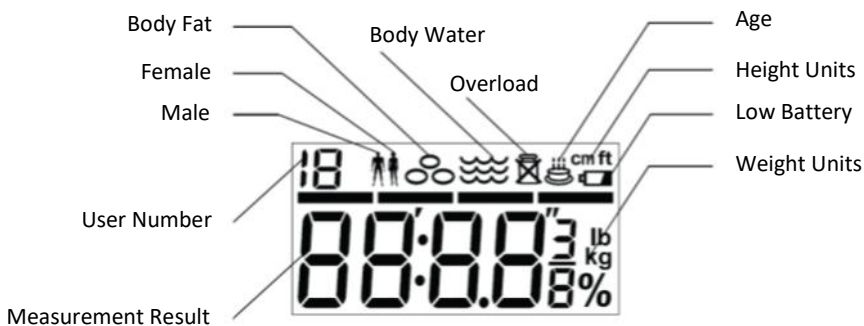

MEASURING BODY WEIGHT & BODY COMPOSITION

1. Press and hold your foot on the center of the scale's platform. The scale will begin cycling through all profile numbers "01" to "10". Lift your foot off the platform of the scale when your profile number is on display to select your profile.
2. After the scale quickly shows your user profile data it will display zero weight. You may now step on the scale and stand still in the center of the platform with clean and dry bare feet, touching each of the metal foot placers. After a few seconds the scale will display your body weight.
3. Next, the scale will display a progress bar while it calculates your body composition. Keep still on the scale with your bare feet contacting the metal foot placers.
4. Finally, the scale will display your current body composition measurements. You may now step off the scale. The scale will cycle through your body weight, body fat and body water measurements four times before turning itself off.

MEASURE BODY WEIGHT WITHOUT A PROFILE (WEIGHT ONLY MODE)

Tap the platform of the scale and remove your foot immediately. After the scale displays "8888" it will show zero weight. You may now step on the scale to measure body weight only. You do not need a user profile saved in the scale. You do not need to be bare foot and your feet do not need to contact the foot placers.

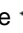
HOW DOES THE SCALE MEASURE BODY COMPOSITION?

This scale uses Bioelectrical Impedance Analysis (BIA) technology to measure body composition. BIA technology sends an unperceivable electronic signal through your body to calculate your body fat, body water and muscle mass percentages. Do not use this scale if you have a pacemaker or other internal electronic medical device.

HOW DO I GET THE MOST ACCURATE RESULTS?

Hydration levels fluctuate throughout the day and are affected by eating, drinking and exercise. As a result, for maximum accuracy and repeatability, it is recommended that this scale is used at the same time of day under the same conditions to establish a personal average. These readings are a close indication, and not an exact value. Please consult your doctor for more information.

REVIEWING USER PROFILE DATA

To view the personal data of your user profile, press the  button with the scale turned off and use the  or  arrow buttons to select a user profile. Follow steps 3 through 7 of the "INITIAL SETUP: CREATING & EDITING USER PROFILES" section located on page two to review, edit and confirm personal profile data.

MEASUREMENTS DURING WEIGHT LOSS

During weight loss, body fat percentages may not change as expected since the reading is a percentage of body fat compared to body weight, and both could be changing at the same time. To minimize the loss of lean body tissue, be sure to include exercise in your weight loss plan.

BODY FAT PERCENTAGE — SUGGESTED RANGES

Age	Men			
	Underfat	Healthy	Overfat	Obese
< 40	< 8%	8-20%	21-25%	> 25%
40-59	< 11%	11-22%	23-28%	> 28%
> 59	< 13%	13-25%	26-30%	> 30%

Age	Women			
	Underfat	Healthy	Overfat	Obese
< 40	< 21%	21-33%	34-39%	> 39%
40-59	< 23%	23-35%	36-40%	> 40%
> 59	< 24%	24-36%	37-42%	> 42%

BODY WATER PERCENTAGE — SUGGESTED RANGES

Age	Men		
	Low	Normal	High
10-15	< 58%	58-72%	> 72%
16-30	< 53%	53-67%	> 67%
31-60	< 47%	47-61%	> 61%
61-80	< 42%	42-56%	> 56%

Age	Women		
	Low	Normal	High
10-15	< 57%	57-67%	> 67%
16-30	< 47%	47-57%	> 57%
31-60	< 42%	42-52%	> 52%
61-80	< 37%	37-47%	> 47%

PROPER USE & CARE

- Use and stow the scale at room temperature.
- Do not submit the scale to shaking, excessive vibration, or other rough treatment; use on a flat and stable surface.
- The display may be affected by electromagnetic disturbances such as radios, microwaves, or cell phones. If such disturbances occur, remove the source of the disturbance, and restart the scale.
- Disassembling, or tampering with the scale voids the warranty.
- Clean the scale with a damp cloth; do not submerge in liquid; scale is not waterproof.
- Failure to insert batteries in the correct polarity, as indicated in the battery compartment, may shorten the life of the batteries or cause batteries to leak.
- Do not mix Alkaline, Standard (Carbon-Zinc) or Rechargeable (Nickel Cadmium) or (Nickel Metal Hydride) batteries. Do not mix old and new batteries.
- Batteries should be recycled or disposed of as per state and local guidelines. Do not dispose of batteries in fire.

TROUBLESHOOTING WARNING MESSAGES

“Err2” — BIA analysis failure / Sensor defect

Solution: Let the scale turn off. Step back on the center of the platform with clean and dry bare feet with both feet on the metal foot placers.

“8888” — Startup cycle stuck

Solution: Let the scale turn off and try measuring again. If the message is persistent and prevents measurements, contact Escali for further instructions.

“- - -” — Unstable surface

Solution: Ensure the scale is on a hard, flat, level and smooth surface that is free of any vibrations.

PRODUCT SPECIFICATIONS

Model: BFBW200

Capacity: 440 lb / 200 kg

Increments: 0.2 lb / 0.1 kg

Product Dimensions: 14 x 14 x 2 in / 35 x 35 x 5 cm

Power Source: Two CR2032 batteries (included)

FCC USER GUIDE INFORMATION

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna;
- Increase the separation between the equipment and the receiver;
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected;
- Consult the dealer or an experienced radio / TV technician for help.

Radio Transmitters (Part 15)

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Caution

The Escali BFBW200 Body Fat / Body Water Bathroom Scale offers you a seamless way to manage your health. Please be aware that this device is designed for adults' self-measuring and self-monitoring body fat and hydration levels. Any information provided by this device is in no way meant to treat, cure or prevent any disease or illness from happening. This device should not be used by anyone who is acutely or chronically ill, suffering from a disease or taking medications that affect your water levels. The accuracy of readings for these patients has not been verified. Specific medical advice should be obtained from a physician.

- Before diet and exercise, you must first find a professional medical guide.
- The Escali BFBW200 is equipped with data transmission function. It may emit electromagnetic energy so as to perform its intended function. Nearby portable and mobile RF communications equipment can affect the performance of the device.
- Portable and mobile RF communications equipment can affect the measuring accuracy of the Escali BFBW200.
- Kindly note that the use of accessories, transducers or cables other than those specified, with internal components, may result in increased EMISSIONS or decreased IMMUNITY of the device.
- Be aware that misuse of electrical equipments can cause electric shock, burns, fire and other hazards. Please do not use the device in a flammable gas environment.
- Warning that the Escali BFBW200 should not be used adjacent to or stacked with other equipment.
- Manufacturer will make available on request circuit diagrams, component parts list, etc.

- **WARNING:** No modifications of this equipment is allowed.
- During using the scale, the patient will contact with the surface and the electrode of the scale. The surface and the electrode of the scale is type BF Applied part.
- The device doesn't need to be calibrated in five years lifetime.
- Do not touch the output of the batteries when using the analyzer.
- The patient is an intended operator. The patient can perform all the operations in the manual, such as measurement, data transmitting, changing batteries.

Product Name	BFBW200 Body Fat / Body Water bathroom Scale
Dimension	Scale: 14 x 14 x 2 in / 350 x 350 x 50 mm (Approximately)
Net Weight	5.18 lb / 2.35 kg (Approximately)
Display	Digital LED
Measurement Unit	Pound (lb) / Kilogram (kg) / Stone (st)
Measurement Range	11 lb to 440 lb / 5 kg to 200 kg / 0 st: 11 lb to 31 st: 6 lb
Division	0.2 lb / 0.1 kg / 0 st: 1 lb
Accuracy	50kg: ±0.3kg; 100kg: ±0.4kg; 150kg: ±0.5kg; 200kg: ±0.7kg
Working Environment	Temperature: 41°F to 104°F / 5°C to 40°C; Relative Humidity: ≤90% RH; Atmospheric Pressure: 86 kPa to 106 kPa
Storage Environment	Temperature: -4°F to 140°F / -20°C to 60°C; Relative Humidity: ≤90% RH; Atmospheric Pressure: 50 kPa to 106 kPa
Power Source	6V (Two CR2032-size Batteries)
Turn on Method	Tap On Technology
Auto-OFF	The scale will turn off after about 15 seconds if there is no operation.
Accessories	1. Two CR2032-size batteries; 2. User Manual
Mode of Operation	Continuous Operation
Degree of Protection	Type BF applied part
Software Version	A0
Protection against Ingress of Water	IP21

	Symbol for "THE OPERATION GUIDE MUST BE READ"		Symbol for "MANUFACTURER"
	Symbol for "TYPE BF APPLIED PARTS"		Symbol for "DIRECT CURRENT"
	Symbol for "SERIAL NUMBER"		Symbol for "ENVIRONMENT PROTECTION—Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local authority or retailer for recycling advice."
	Symbol for "MANUFACTURE DATE"		

ESCALI CORPORATION

3203 Corporate Center Drive, Suite 150
Burnsville, MN 55306
USA

PRODUCT SUPPORT

24/7 Online support: <https://escali.zendesk.com>

Email support: feedback@escali.com

Phone support: 1-800-467-6408

LIFETIME LTD WARRANTY

This scale is guaranteed to be free of defects in materials and workmanship. For more details visit: www.escali.com/warranties

Escali's liability is limited to two (2) times the cost of the product.

© Escali Corp., Minneapolis, MN, USA. All rights reserved. Escali, the Escali logo and other Escali marks are owned by Escali Corp. and may be registered.

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GARANTIE À VIE LTD.